TOKUYAMA REBASE III

ENGLISH

Carefully read all information, precautions, notes and materials before using TOKUYAMA REBASE III.

PRODUCT DESCRIPTION AND GENERAL INFORMATION

1. TOKUYAMA REBASE III is an acrylic resin used for chairside hard denture linings and extension of denture borders.
2. The POWDER contains Poly(ethyl methacrylate) and Benzoyl peroxide. The LIQUID contains methacrylate monomer. REBASE ADHESIVE contains ethyl acetate, acetone and methacrylate polymer.
3. REBASE ADHESIVE is used solely for bonding to the acrylic surfaces. It cannot be used to bond nylon, metal, and silicone surfaces, etc.
4. TOKUYAMA REBASE III kit contains POWDER, LIQUID, REBASE ADHESIVE, TOKUSO RESIN HARDNER II and other accessories such as measuring cup, dropper, rubber cup, spatula, drip nozzle, drip cup, brush, and spoon.

*Limited availability of FAST SET and/or NORMAL SET in certain areas. Contact your local dealer for availability.

INDICATIONS

1. Relining of dentures
2. Extension of denture borders

CONTRAINDICATIONS

TOKUYAMA REBASE III contains methacrylic monomer. DO NOT use TOKUYAMA REBASE III for patients allergic to or hypersensitive to methacrylic monomers and related monomers or to any of the other ingredients.

PRECAUTIONS

1. DO NOT use TOKUYAMA REBASE III for any purpose other than those listed in these instructions. Use TOKUYAMA REBASE III only as directed herein.
2. TOKUYAMA REBASE III is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
3. DO NOT use TOKUYAMA REBASE III if the safety seals are broken or appear to have been tampered with.
4. If TOKUYAMA REBASE III causes an allergic reaction or hypersensitivity, remove the prosthesis and discontinue its use immediately.
 - Before using TOKUYAMA REBASE III, perform tissue conditioning for one week if there are signs of inflammation of the mucous membrane, or tissue irritation from old denture.
 - Discontinue wearing the newly relined denture if there are any signs of hypersensitivity, redness, and ulcerations.
5. Use examination gloves (plastic, vinyl or latex) at all times while handling TOKUYAMA REBASE III to avoid the possibility of allergic reactions from methacrylic monomers.

Note: Certain substances/materials may penetrate through examination gloves. If TOKUYAMA REBASE III comes in contact with the examination gloves, remove and dispose of the gloves and wash hands thoroughly with water as soon as possible.
6. Avoid contact of TOKUYAMA REBASE III with eyes, skin and clothing.
 - If TOKUYAMA REBASE III comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If TOKUYAMA REBASE III comes into contact with the skin or clothing, immediately saturate the area with an alcohol-soaked cotton swab or gauze and flush with water.
 - Whenever possible, avoid contact of TOKUYAMA REBASE III with oral mucous membrane other than the region to be lined.
 - Instruct the patient to rinse his mouth immediately after treatment.
7. DO NOT allow excess materials to flow down the patient's trachea or esophagus. Aspirated or ingested TOKUYAMA REBASE III can cause asphyxiation or other injury. Immediately remove any excess lining material from the palatal area to avoid materials flowing down the patient's throat. TOKUYAMA REBASE III is not radiopaque, and radiographs will not detect aspirated or ingested material. In the case of elderly patient whose reflex ability is compromised, set the chair in an upright position and instruct the patient to breathe through the nose in order to prevent the material from entering the airway.
8. To avoid the unintentional ingestion of TOKUYAMA REBASE III, do not leave it unsupervised within the reach of patients and children.
9. DO NOT expose the LIQUID or its vapor to open flame as the LIQUID is flammable.
10. REBASE ADHESIVE contains ethylacetate and acetone, which are highly flammable and volatile solvent. DO NOT expose REBASE ADHESIVE to open flame. Use in a ventilated room during use and close the cap tightly after use.
11. DO NOT apply REBASE ADHESIVE to the patient's mouth or gums. Application of REBASE ADHESIVE must be performed outside of the patient's mouth.
 - In the case of contact with the mouth, immediately instruct the patient to rinse his/her mouth.
12. DO NOT mix TOKUYAMA REBASE III with any materials not noted in these instructions.
13. Always clean the powder measuring cup, rubber cup, spatula, drip cup, and brush with alcohol after each use to avoid cross contamination.
14. DO NOT return the POWDER, LIQUID or REBASE ADHESIVE to the bottle once it is dispensed from the bottle.
15. To avoid inhaling dust while polishing, always wear a mask recommended by federal and/or local regulations.
16. DO NOT place REBASE ADHESIVE in direct with plastic materials, contact may dissolve some types of plastic.

STORAGE

1. Store TOKUYAMA REBASE III at temperatures between 0-25°C (32-77°F).
2. Keep away from heat, open flame, humidity and sunlight.
3. DO NOT use TOKUYAMA REBASE III after the indicated date of expiration.
4. If LIQUID is not to be used for more than a month, store the LIQUID with the original cap. DO NOT store LIQUID with dropper.

DISPOSAL

1. To dispose of materials, mix the POWDER and LIQUID together, and allow it to harden. Be careful of the exothermic reaction produced by the setting material.
2. Any remaining REBASE ADHESIVE should be absorbed in gauze or cotton, and disposed of in accordance with Federal and/or local regulations. Avoid any inhalation of the volatile substance.

CLINICAL PROCEDURE OF TOKUYAMA REBASE III

(1) Cases of Complete Denture (having no undercut)

1. Checking the adaptation and cleaning denture

Check intra-oral condition by inspection and palpation. Remove denture from patient's mouth, clean and air-dry the denture surface completely.

2. Removal of denture surface

Carefully remove the old resin and expose the new resin of the denture surface, especially the surface of the mucosal side, the marginal, buccal and labial areas where it is necessary to be relined with TOKUYAMA REBASE III.

Note:

- 1) Areas where the TOKUYAMA REBASE III should not contact, such as artificial teeth, the denture base or the inter-dental area, should first receive a coat of a separating agent.
- 2) As the relining surface of denture becomes larger, the resin paste shows a higher resistance in its flow. In this case, remove a large amount of the old resin or make a hole to let the resin flow out.

3. Dispensing the REBASE ADHESIVE

Open the bottle of REBASE ADHESIVE. Attach the drip nozzle to the bottle. Dispense an adequate amount of REBASE ADHESIVE into the drip-cup. Replace the cap, closing the bottle tight immediately after dispensing.

- 1) DO NOT directly place a brush into the bottle of REBASE ADHESIVE.
- 2) DO NOT leave REBASE ADHESIVE in the cup for a prolonged period of time.

4. Applying the REBASE ADHESIVE



Apply REBASE ADHESIVE in a single layer with the brush to all areas of the surface to be relined. Leave REBASE ADHESIVE for 20 seconds at room temperature. If REBASE ADHESIVE liquid is still observed on alveolar crest area of the denture, air-dry until completely dry.

Note:

- 1) Apply TOKUYAMA UNIVERSAL BOND (sold separately) to any metal before applying REBASE ADHESIVE to the resin of the denture if there is metal in the area to be relined.
- 2) Apply REBASE ADHESIVE to the buccal and labial side of the denture to be relined.
- 3) Apply REBASE ADHESIVE twice to the marginal area to ensure good adhesion the denture.
- 4) REBASE ADHESIVE will remain effective for ten minutes after application.
- 5) Apply REBASE ADHESIVE only to denture surface. DO NOT mix REBASE ADHESIVE with POWDER and/or LIQUID.

5. Measuring and mixing

Use the POWDER of every one graduation (1.0) of the measuring cup to the LIQUID with the dropper two graduations (1.0). After measuring, place the POWDER into LIQUID (using the supplied rubber cup) and mix gently with the spatula for 5-20 seconds (10 seconds after the powder is blended into the liquid). Mix gently to minimize bubbles.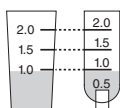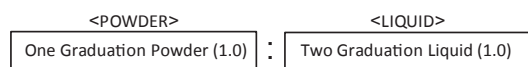

Note:

- 1) Be sure to use powder dedicated cup for TOKUYAMA REBASE III. The scale position of cup is different from that of TOKUYAMA REBASE II.
- 2) FAST and NORMAL come with different powder measuring cups. Be sure to use each powder with its appropriate cup.
- 3) Put the LIQUID in the cup first and then the POWDER to minimize the formation of bubbles.
- 4) Mixing too vigorously or for too long of a period will facilitate bubble formation, producing an uneven consistency. This in turn will create a rough surface and result in incomplete curing.
- 5) If necessary or so desired, the viscosity of the resin can be increased by increasing the quantity of the POWDER used. Beware of adding too much POWDER as the POWDER and LIQUID mix easily, it is best to measure the ratio for optimum results.
- 6) If powder is added during mixing, surface irregularities due to insufficient mixing may occur. Before mixing, measure the amount of liquid and powder used.
- 7) Close the cap tightly, after use of POWDER or LIQUID.
- 8) DO NOT leave POWDER or LIQUID in the cup for a prolonged period of time.

6. Applying the resin

With the spatula, apply the resin paste to all surfaces being relined, especially to the marginal, buccal and labial areas of the denture. If too much time is taken to mix the POWDER and LIQUID, the available working time will be shortened.

Note:

- 1) Apply resin within 20 to 60 seconds after mixing for FAST (for NORMAL, within 30 to 90 seconds).
- 2) Be careful not to apply too much resin to the surface. Applying too much resin to the surface will result in the resin flowing down the patient's throat upon insertion of the denture.
- 3) Viscosity and hardening are temperature dependent. The warmer the room temperature, the faster the set. The colder the room temperature, the slower the set.

7. Insertion of the denture

Insert the denture properly into the patient's mouth and have them bite down correctly at the centric position.

Note:

- 1) Be extremely careful to avoid occlusal malposition.
- 2) Make sure there is no extra resin flowing down the patient's throat.
- 3) In case excess resin flows into the patient's throat, remove it with a fingertip or an appropriate instrument.

8. Muscle treatment

After following the above steps, and when the resin begins to harden, perform the necessary treatment of the muscles in the mouth. Remove the excess portion of the resin intra-orally.

Note:

- 1) Be sure to complete all of the muscle treatment within 1 to 2 minutes from the time mixing started for FAST (90 seconds to 4 minutes for NORMAL).

9. Oral maintenance of the denture (final curing)

Keep the denture in the patient's mouth until final hardening is complete.

Note:

- 1) Keep the denture fixed in the patient's mouth for 2 to 5 1/2 minutes after mixing has commenced for FAST. For NORMAL keep the denture fixed in the patient's mouth for 4 to 8 minutes. The resin will cure at this time. DO NOT remove the denture from the mouth during this period, since the presence of oxygen prevents the resin from fully curing.
- 2) Incomplete cure as a result of removing the denture too early will create undesired problems such as an inadequate surface hardness or a poor adaptation.
- 3) DO NOT cure intra-orally in the situation where patient has an undercut. Treat according to the clinical procedure "Cases of Undercut or Partial Denture" described below.
- 4) In the case of trimming excess portion of the resin, trim according to the clinical procedure "Cases of Undercut or Partial Denture" described below.
- 5) For additional hardness of cured denture reline, TOKUSO RESIN HARDENER II maybe used. TOKUSO RESIN HARDENER II improves the surface hardness and final polish of the denture base.

10. Modifying and polishing

After final hardening, use a carbide bur, silicone point, finishing wheel, etc. to modify and polish the denture.

11. Checking the adaptation

12. Completion

(2) Cases of Undercut or Partial Denture

Treat according to methods 1 - 7, and 10 - 12 of the clinical procedure "Cases of Complete Denture (having no undercut)".

8. Muscle treatment and trimming

After following the above steps, and when the resin begins to harden, perform the necessary treatment of the muscles in the mouth. When the resin becomes harder than paste, remove the denture from the patient's mouth, and trim it extra-orally prior to final curing.

Note:

- 1) Be sure to complete all of the muscle treatment and trimming within 2 to 3 minutes from the time mixing started for FAST (4 to 5 1/2 minutes for NORMAL).
- 2) In the situation where patient has an undercut, partial denture, residual root or other problems, remove the denture from the mouth and trim it extra-orally prior to final curing. Otherwise it will be difficult to remove the denture from the mouth.

9. Extra-oral maintenance of the denture (final curing)

Final cure extra-orally, using TOKUSO RESIN HARDENER II.


<Application of TOKUSO RESIN HARDNER II >

- 1) Prepare 40-60°C (104-140°F) water in a container. Be sure to pour enough water to completely immerse the denture.
- 2) Dissolve 1 scoop of TOKUSO RESIN HARDENER II per 200mL of water.
- 3) Once the material is completely dissolved, immerse the denture for 3 minutes.
- 4) Remove the denture, rinse, and polish.


Note:

DO NOT mix TOKUSO RESIN HARDNER II with POWDER and/or LIQUID.

<STANDARD TREATMENT TIME for FAST>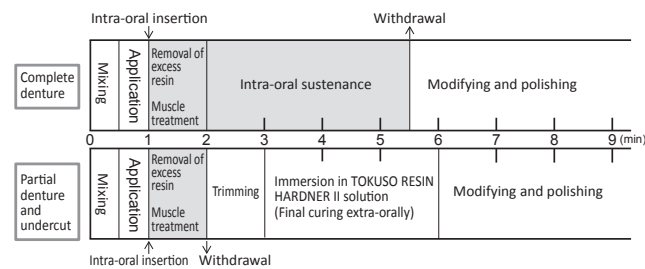

<STANDARD TREATMENT TIME for NORMAL>

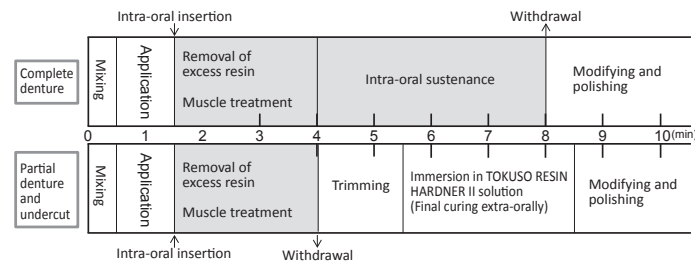Note:

Standard treatment time is based on temperatures of 23°C/73°F (Extra-oral) and 37°C/98°F (Intra-oral). Treatment time is shortened by a rise in body or room temperature and as the proportion of powder increases.

CAUTION: Federal law restricts this device to sale by or on the order of a dental professional.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the dental professional to ensure this product is suitable for application before use.

Specifications are subject to change without notice. When the product specifications change, the instructions and precautions may also change.

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